

Whiff!®

Hygienic Cleaner & Deodorizer Spray - Healthcare Use Instructions

IMPORTANT – READ BEFORE USE

- This product is a hygienic cleaner and deodorizer.
- Not a sterilant. Not a high-level disinfectant.
- For use on hard, non-porous surfaces and external surfaces of non-critical equipment only.
- Follow all facility policies, infection prevention guidance, and equipment manufacturer (OEM) instructions.

Definition – Non-Critical Equipment:

Items that contact intact skin only and do not penetrate skin or mucous membranes (e.g., IV poles, carts, wheelchairs, equipment exteriors).

1. INTENDED USE: Whiff! is intended for routine cleaning and deodorizing of environmental surfaces and external surfaces of non-critical equipment in healthcare settings, including patient rooms, treatment areas, and common spaces.

2. PURPOSE: Whiff! supports routine environmental hygiene and odor control by providing a pH-neutral, water-based cleaning option for surfaces where disinfection is not required. Whiff! complements existing facility cleaning and disinfection protocols by enabling appropriate product selection based on task, surface type, and clinical risk.

3. APPROVED USE AREAS (Examples)

- Bed rails, over-bed tables, chairs
- Bedside cabinets, counters, desks
- Door handles, light switches
- Bathroom fixtures (sinks, toilets, grab bars – exterior surfaces)
- Waiting room furniture
- Glass and mirrors (e.g., windows, mirror surfaces)
- External surfaces of non-critical equipment (per OEM guidance)

4. DO NOT USE ON

- Invasive or implantable medical devices
- Inside equipment housings, vents, ports, or sealed electronics
- Open wounds, broken skin, eyes, or mucous membranes
- Glassware used for drinking or food consumption
- Any surface prohibited by the equipment manufacturer

5. PERSONAL PROTECTIVE EQUIPMENT (PPE)

- Disposable gloves recommended
- Use in a ventilated area

6. HOW TO USE – ROUTINE CLEANING & DEODORIZING

Step 1 – Pre-Clean (If Needed)

- Remove visible soil per facility policy

Step 2 – Apply

- Preferred: Spray onto a clean disposable cloth (recommended for equipment and glass)
- Or spray directly onto the surface from 6–12 inches

Whiff!®

Hygienic Cleaner & Deodorizer Spray - Healthcare Use Instructions

6. HOW TO USE – ROUTINE CLEANING & DEODORIZING (Continued)

Step 3 – Wipe

- Wipe evenly; lightly wet the surface
- Wipe in one direction when possible

Step 4 – Dry

- Allow to air dry or wipe dry if required

Step 5 – Finish

- Dispose of wipes per policy
- Perform hand hygiene

7. HOSPITAL EQUIPMENT – EXTERIOR CLEANING

- Use only on equipment tolerant of incidental moisture
- Turn off/unplug if required
- Spray onto cloth only; do not spray directly onto equipment
- Wipe screens, cables, rails, housings, and touch points
- Avoid liquid entry into seams/openings
- Allow to dry before use

8. ODOR CONTROL USE

- Remove odor sources first
- Clean affected hard surfaces
- Repeat for high-touch areas as needed

9. CHEMICAL SAFETY

- Do not mix with other cleaners or disinfectants
- Avoid contact with acids or acidic cleaners

10. STORAGE & HANDLING

- Store at room temperature
- Keep bottle closed when not in use
- Do not freeze or expose to heat or direct sunlight

11. FIRST AID & INCIDENTAL CONTACT

- Eyes: Rinse with water if contact occurs.
- Skin: *Incidental skin contact is not expected to cause irritation.* If contact occurs wipe product from skin or rinse with water.
- Inhalation discomfort: Move to fresh air.
- Ingestion: Rinse mouth; seek medical advice.