

TM

Diversey has been, and always will be, a pioneer and facilitator for life. We constantly deliver revolutionary cleaning and hygiene technologies that provide total confidence to our customers across all of our global sectors.

Diversey is headquartered in Fort Mill, SC, USA. For more information, visit [www.diversey.com](http://www.diversey.com) or follow us on social media.



**Sanitis**  
**Austin, TX**  
**(888) 789-6266**  
**[www.SanitisGlobal.com](http://www.SanitisGlobal.com)**

© 2020 Diversey, Inc. All Rights Reserved. 39481-LIT-en (20/010)

**CLEANPatch®**

**Medical surface repair patch**  
for hospital beds and stretchers



## The Problem...

Damaged mattresses are common in healthcare facilities, and have been shown to harbor pathogens that contribute to Healthcare Associated Infections.

Studies in Canada, the US and UK reveal that 26% to 48% of mattresses in circulation are damaged.\*

CLEANPatch is a registered trademark of Surface Medical, Inc., used under license.

## The Solution is Here...

### Previous Alternatives

Replace Mattress	<ul style="list-style-type: none"> <li>✓ Validated</li> <li>✗ Unaffordable</li> <li>✗ Inconvenient</li> </ul>
Repair with Tape	<ul style="list-style-type: none"> <li>✓ Inexpensive</li> <li>✗ Unproven</li> <li>✗ Not Durable</li> </ul>
Do Nothing	<ul style="list-style-type: none"> <li>✗ Risk Patient Safety</li> </ul>



### Why CleanPatch®?

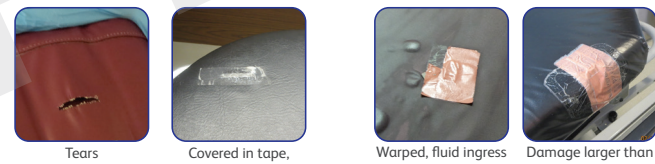
**CleanPatch® is the first medical surface repair patch for hospital beds and stretchers. It restores damaged mattresses to an intact and hygienic state.**

- Clinically Tested**
- Validated by independent Infection Prevention Professionals
  - Durable and impervious to fluids under rigorous conditions
  - Fully cleanable with hospital disinfectants
- Safe and Simple**
- Early intervention reduces the risk of cross contamination from damaged mattresses
  - Made of latex-free, biocompatible materials
  - Simple peel and stick application, adheres in seconds
- Cost Effective**
- Extends the life of mattresses and stretchers
  - Proactively improves patient safety
  - Significant savings from equipment repair versus replacement

### Getting Started:

1. Determine who will manage CleanPatch® applications in each clinical area.
2. Inspect mattresses during, or after, every terminal cleaning.
3. Ensure CleanPatch® is readily available at the point of use.
4. Review the CleanPatch® application instructions and Implementation Guide.
5. Keep track of repairs to monitor the durability and the savings.

## Use CleanPatch® as part of a Mattress Inspection and Repair program

	✓ Use CleanPatch®	✗ Do Not Use CleanPatch®
<b>Mattress Age and Condition</b>	Usable within expected lifespan	No longer suitable for use
<b>Damage Size and Location</b>	Less than 4 inches (10 cm) Flat areas or over edges (2-D)	More than 4 inches (10 cm) Sharp corners (3-D)
<b>Fluid Penetration</b>	No signs of fluid ingress	Mattress cover is visibly stained or warped
<b>Size of Damage</b>	Use 1 patch per damage area	Layering multiple patches not recommended
<b>Case Examples</b>	 <div> <div>Tears</div> <div>Covered in tape, repairable</div> <div>Warped, fluid ingress</div> <div>Damage larger than four inches</div> </div>	

## Product Information

Product #	Shape	Shape	Damage Size*
#11003	Round	2 inch (5 cm)	Up to 1/2 inch (1.3 cm)
#10003	Square	3.5 x 3.5 inch (9 x 9 cm)	Up to 2 inches (5 cm)
#12003	Rectangle	3.5 x 6 inch (9 x 15 cm)	Up to 4 inches (10 cm)
#18003	Mixed Box	2 inch (5 ea.) 3.5 inch (3 ea.) 3.5 X 6 inch (2 ea.)	See references above



To ensure product integrity,  
**Do Not cut CleanPatch®**